
Designed in the Netherlands
Made in the EU

www.push.eu
The push
6199 AB Maatschappij-Airport
Entrepotlaan 31
Netherlands
The push
2.20.1
REF

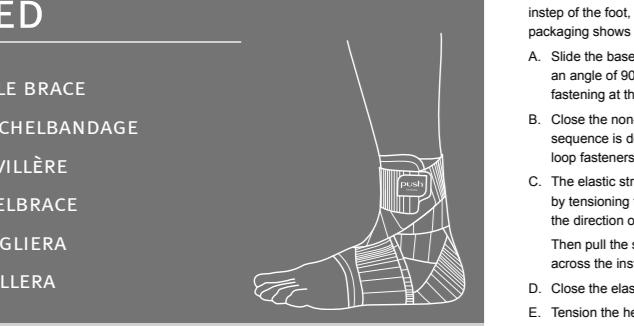PRODUCT INFORMATION

EN • DE • NL • FR • IT

ES • PT • CS • PL • HU

JA • SV • DA • NO • FI

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, we refer to www.push.eu/distributors

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PUSH MED ANKLE BRACE

INTRODUCTION

Push products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus. Approved by physicians and scientists. Your new ankle brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med ankle brace increases ankle stability and helps control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med ankle brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

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The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med ankle brace is easy to apply and can be washed without any problem.

INDICATIONS

- Moderate to serious instability
- Acute lateral ankle ligament trauma
- Stable ankle fracture
- Follow-up treatment of ankle fractures
- Postoperative treatment of ligament injuries or fractures
- Capsular irritation
- Tendonitis
- Arthritis with instability
- Secondary prevention of ankle ligament trauma
- Secondary prevention of ankle ligament trauma
- Impingement (capsular clamping; anterior and posterior)
- Football ankle

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Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINATION SIZE / FITTING

The ankle brace is available in 5 sizes. Position the foot at an angle of 90° in relation to the lower leg. Then measure the circumference around the heel and the instep of the foot, as shown in the illustration on the packaging. The table on the packaging shows the correct size for your foot.

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WICHTIG

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly. Always consult your physician or therapist if complaints persist or deteriorate.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of Nea International bv

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